

System Overview



- UniSUC System is an innovative solution to managing adult urinary incontinence
- Engineered for comfort, ease of use, adjustability to fit small-to-large body frames, moisture reduction at skin contact areas with highly effective wicking methodology that reduces Catheter Acquired Urinary Track Infections (CAUTIs), and lower cost of ownership.



UniSUC Frame Device
Part #: **22-F**



Disposable Liner
Part #: **22-L** [10-pack]

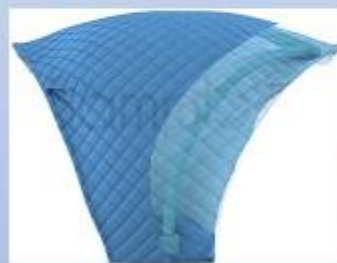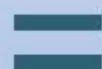


UniSUC Assembly



Collection Container
& Vacuum Source
[not included]

Unisex


UniSUC can be used by Females or Males

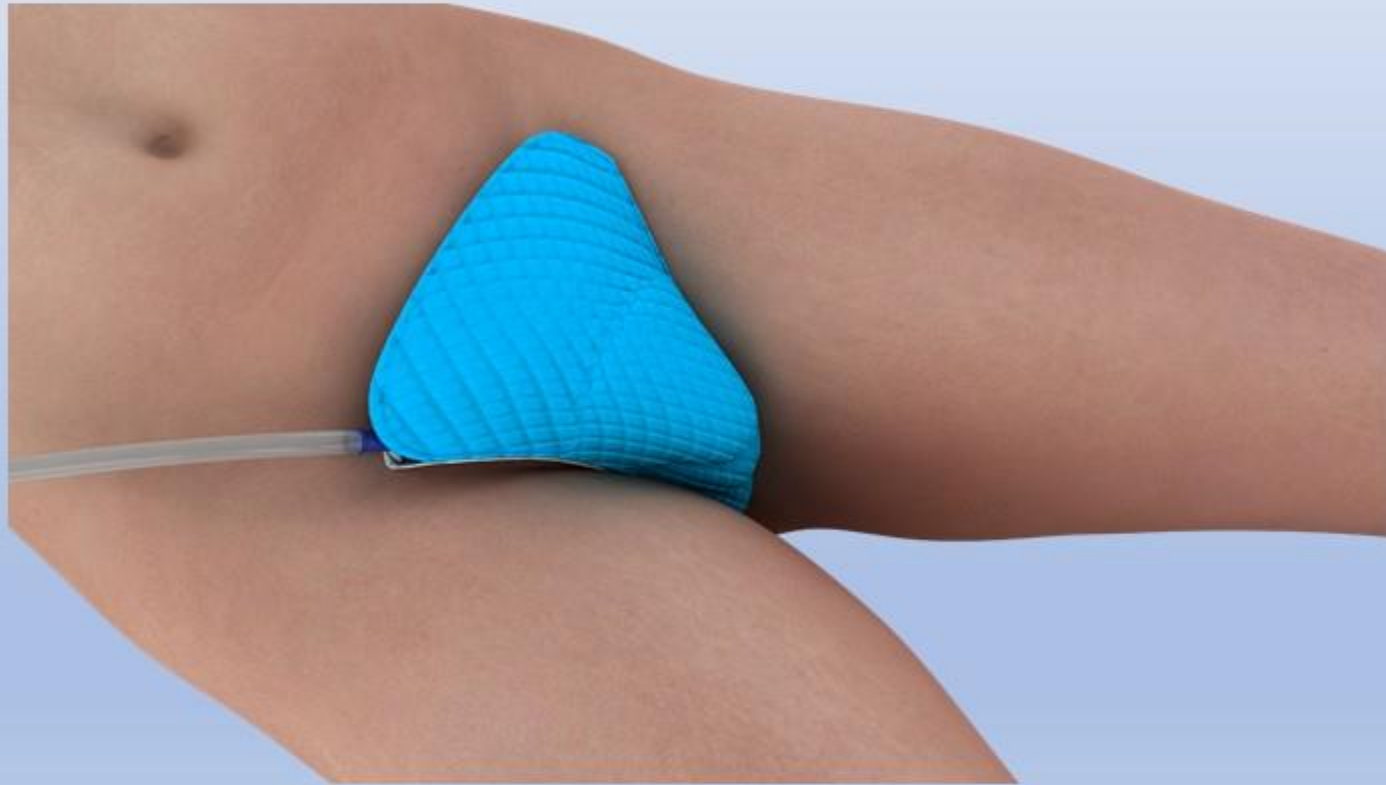

Requirements (not included with Package)

Assembly requires a standard Collection Container & Vacuum Source / Suction Device (not included):

- (A) Collection Container & Vacuum Source
- (B) Suction Tubing

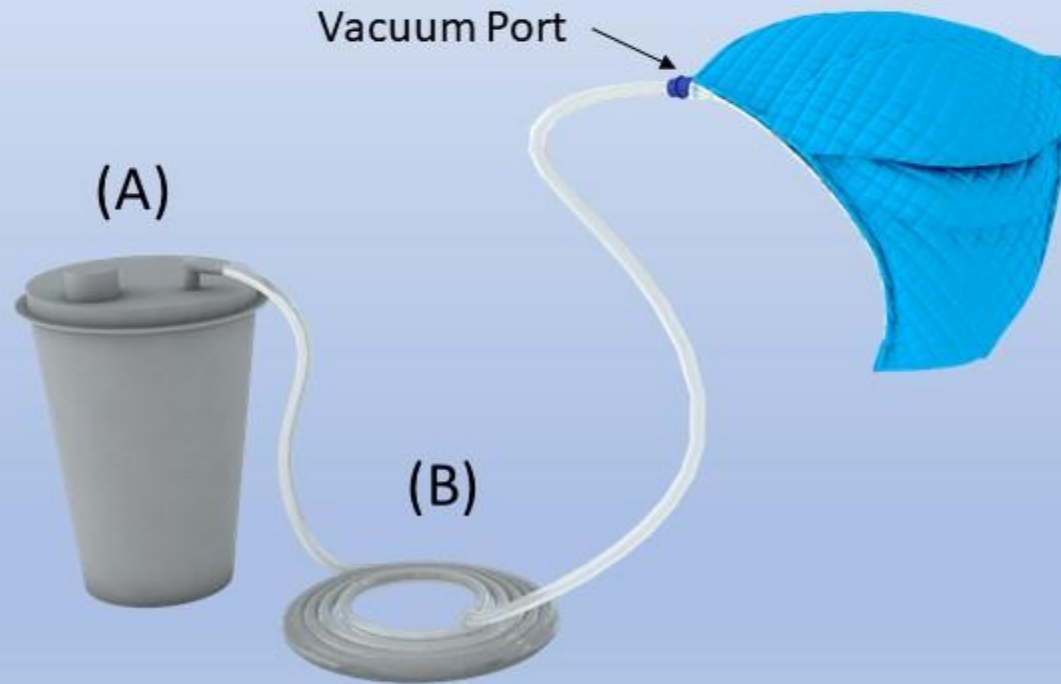

Patient Preparation

- Verify hospital protocol for patient order of use.
- Assess patient perineal skin area as standard practice protocol.
- Patient to be lying in bed or sitting positions.

Easy Installation – Step 1


Patient Fitting - Custom fit UniSUC Frame Device to patient



- ✓ Lay patient on back or sitting position with legs slightly open
- ✓ Place narrow end (tip) of Frame Device below vagina or scrotum above anus
- ✓ Observe gaps along the contour shape of the body frame




- ✓ Remove Frame Device & bend to match its shape to the contours of the patient's body shape around the genital area

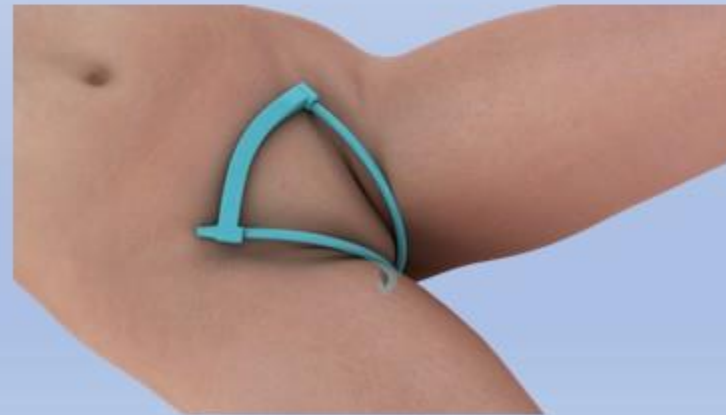

- ✓ Verify match by placing narrow end (tip) of Frame Device below vagina or scrotum, above anus, again
- ✓ Observe gaps along the contour shape of the body and frame
- ✓ Repeat process if any significant gaps are observed


Easy Installation – Step 2

UniSUC assembly - Combine Frame Device with Disposable Liner and attach suction tube to vacuum port of UniSUC Frame Device

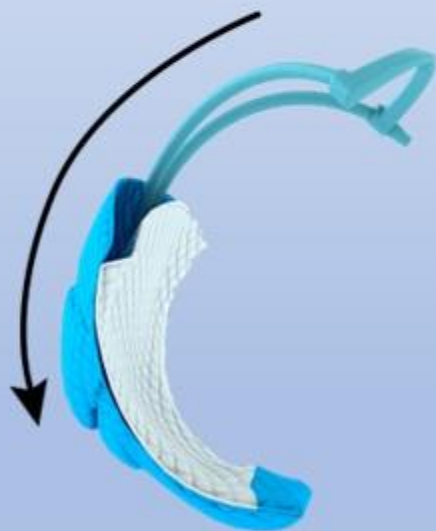

- ✓ Remove Frame Device from patient
- ✓ Insert entire Frame Device into one of the Liners provided in the kit
- ✓ Ensure proper orientation; light blue Plastic Barrier layer should be on the outer side of the bend, with soft absorbent material inside of bend

- ✓ Attach suction tube from Suction Device Collection Container to the vacuum port

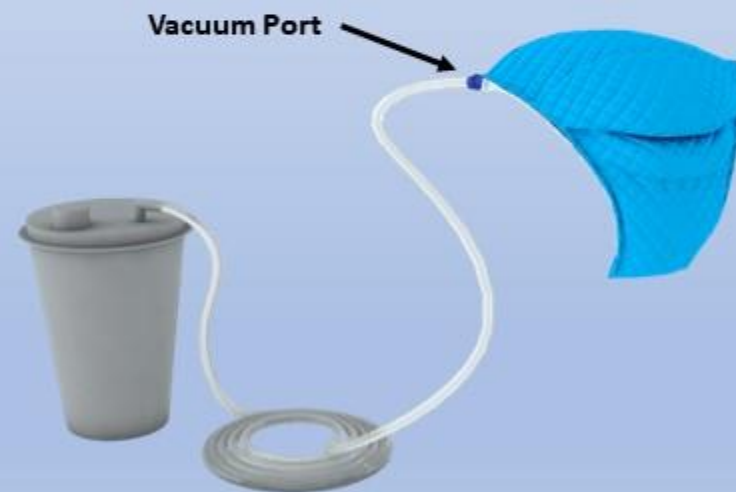

Easy Installation – Step 3

Apply UniSUC - Install UniSUC assembly onto patient and switch on Vacuum Source / Suction Device



- ✓ Place narrow end (tip) of COMBINED Frame & Liner below vagina or scrotum
- ✓ Press UniSUC System firmly onto the body
- ✓ Adjust for comfort

- ✓ Double check suction tube connection to UniSUC vacuum port is secure
- ✓ Switch ON Suction Device for continuous suctioning (minimum 40mmHg of vacuum for best results)